Orthodontic Elastomeric Ligature Ties

Intended Use

Ligature ties are intended to be placed around the tie wings on the bracket to retain the archwire in the bracket slot.

Description of the device

Elastomeric ligature ties (Patient Ties) are small, toric-shaped elastomers. Elastomerics are available attached to a handle, placed on a cane or in a bulk bag.

Elastomeric ligatures are malleable and easily place around an orthodontic bracket. They are also referred to as “o-rings,” “donuts,” “elastomerics” and “lig ties.”

1. Part Numbers

Product	Part	Variants	Material	Composition
Ligature Ties Polyurethane	BT* OCM* VT* OCS*	Handle style and color	Texin 285	Thermoplastic Polyurethane
Designer Ties	DTIE* DT*	Handle style and color	Texin 285	Thermoplastic Polyurethane
Ligatures Polyurethane	BCL* BGY* CCL* CGY*	Size, color Bulk, Caned	Texin 285	Thermoplastic Polyurethane

Warnings and Precautions

	Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	Ligature breakage and or dislodgement requires immediate attention by the orthodontist.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or aspiration, instruct the patient to seek medical care.
	Care should be taken to avoid habits or foods that could break or dislodge the ligature tie. Follow orthodontist recommendations for proper at home care.

Clinical Benefits

Elastomeric ligatures, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

Orthodontic treatments are subject to relapse. Elastomeric ligatures, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications.

Potential Side Effects or Adverse Events

- Discomfort and pain associated with wire engagement
- Gingival impingement
- Ingestion
- Aspiration
- Elastomeric ligatures, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis

Instructions for use

The use of elastomeric ligatures is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

1. Select the elastomeric ligature to be used. Consider patient preferences when choosing color.
2. Place the elastomeric ligature over the tie wings of the bracket until the desired securement has been achieved. Inadequate securement of the wire may necessitate use of a stainless steel ligature.
3. Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacture and the competent authority.

Maintenance and Storage instructions

Store elastomeric ligature ties in product packaging until use.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions


	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care.
	Elastomeric ligature breakage and or dislodgement requires immediate attention by the orthodontist.
	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Care should be taken to avoid habits or foods that could break or dislodge the Elastomeric ligature. Follow orthodontist recommendations for proper at home care.

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in

or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

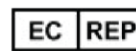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