Orthodontic Buccal Tubes

Intended Use

Tubes are intended to be affixed to the teeth, molar bands or slid onto the archwire to provide an anchor for a wire or other appliance.

Description of the device

An orthodontic tube is an anchoring mechanism available in different prescriptions and slot sizes with design features specific to each molar. Tubes are available as single, double or triple tubes, all having hooks incorporated to the main framework. Tubes are available as convertible or non-convertible and bondable or weldable.

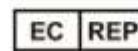
1. Part Numbers

Product	Part	Variants	Material	Composition
Viper Non-Convertible Buccal Tube	NS3B* NS4B*	Prescription; Right, Left; Slot height;	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Viper Convertible Single Buccal Tubes	CS3B*	Roth, MBT; Right, Left; Slot height	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Magnum Single Buccal Tube Convertible	CS1B*	MBT, Roth; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Convertible HG Occlusal Buccal Tube	CC1B*	MBT, Roth; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Double Convertible Buccal Tube	CD1B	MBT, Roth; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Non-Convertible Buccal Tube	NC1B*	Prescription; Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Rickets Triple Non-Convertible Buccal Tube	NRT1B*	Right, Left	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Rickets Double Non-Convertible Buccal Tube	NRD1B*	Right, Left	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Single Non-Convertible Buccal Tubes	NS1B*	Prescription; Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Mini Buccal Tube	NS2B*	Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%

Magnum Triple Convertible Buccal Tube	CT1B	Roth, MBT; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Viper Combination Convertible Buccal Tube	CC3B* CL1B*	Right, Left; Slot height	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
MiniPrevail Bondable Buccal Tube 4.2 MM	NS6B*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
MiniPrevail Bondable Buccal Tube 3.2 MM	NS7B*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Magnum Combination Convertible HG Gingival Buccal Tube	CC1B* CCB* CL1B*	Roth, MBT; Right, Left; Slot height;	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Agility Single Non-Convertible Buccal Tube	AS1B*	Prescription; Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Viper Non-Convertible Buccal Tube	NS3B* NS4B*	Prescription; Right, Left; Slot height;	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Viper Convertible Single Buccal Tubes	CS3B*	Roth, MBT; Right, Left; Slot height	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Magnum Single Buccal Tube Convertible	CS1B*	MBT, Roth; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Convertible HG Occlusal Buccal Tube	CC1B*	MBT, Roth; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Double Convertible Buccal Tube	CD1B	MBT, Roth; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Non-Convertible Buccal Tube	NC1B*	Prescription; Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Rickets Triple Non-Convertible Buccal Tube	NRT1B*	Right, Left	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Rickets Double Non-Convertible Buccal Tube	NRD1B*	Right, Left	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Single Non-	NS1B*	Prescription;	316 Stainless	Iron 60%; Chromium

Convertible Buccal Tubes		Right, Left; Slot height	Steel	18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Mini Buccal Tube	NS2B*	Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Triple Convertible Buccal Tube	CT1B	Roth, MBT; HG I.D.; Right, Left; Slot height	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Viper Combination Convertible Buccal Tube	CC3B* CL1B*	Right, Left; Slot height	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
MiniPrevail Bondable Buccal Tube 4.2 MM	NS6B*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
MiniPrevail Bondable Buccal Tube 3.2 MM	NS7B*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Magnum Combination Convertible HG Gingival Buccal Tube	CC1B* CCB* CL1B*	Roth, MBT; Right, Left; Slot height;	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Agility Single Non-Convertible Buccal Tube	AS1B*	Prescription; Right, Left; Slot height	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Viper Non-Convertible Buccal Tubes	NS3*	Prescription; Right, Left; Slot height; Weldable, Bondable	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Viper CV Convertible Single Buccal Tubes	CS3*	Prescription; Right, Left; Slot height; Weldable, Bondable	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Magnum Single Buccal Tube Convertible buccal Tubes	CS1* CS1B*	Prescription; Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Convertible HG Occlusal Buccal Tube	CC1*	HG I.D.; Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Double Convertible Buccal Tube	CD1*	Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Combination Non-Convertible Buccal Tube	NC1*	Prescription; HG I.D.; Right, Left; Slot height; Weldable,	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%

		Bondable		
Magnum Rickets Triple Non-Convertible Buccal Tube	NRT1*	Right, Left; Weldable, Bondable	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Rickets Double Non-Convertible Buccal Tube	NRD1*	Right, Left	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Single Non-Convertible Buccal Tubes	NS1*	Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Magnum Triple Convertible Buccal Tube	CT1*	Roth, MBT; HG I.D.; Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
Tubes-Round	M061*			Iron Balance; Nickel 0-15%; Chromium 11.5-20%; Silicon 0-2%; Manganese, 0-2%; Molybdenum 0-6.5%; Titanium 0-0.7%; Copper 0-5%; Aluminum 0-4%; Niobium 0-0.6%; Carbon 0-1.2%; Tantalum 0-0.5%
Viper Combination Convertible Buccal Tube	CC3*	Right, Left; Slot height; Weldable, Bondable	17-4 pH, 302 Stainless Steel	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Weldable Lingual Sheath	M040*		SS Alloys	Chromium 17-20%; Nickel 8-13%; Manganese <2%; Molybdenum max 2.5%
Magnum Combination Convertible HG Gingival Buccal Tube	CC* CC1* CL1*	Roth, MBT; Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%
MiniPrevail Weldable Buccal Tube 4.2MM	NS6*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
MiniPrevail Weldable Buccal Tube 3.2 MM	NS7*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
HODI Buccal Tube	HODI*	Prescription; Right, Left; Slot height	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%
Agility Single Non-Convertible Buccal Tube	AS1*	Prescription; Right, Left; Slot height; Weldable,	17-4 pH	Iron 75%; Chromium 16.2%; Nickel 4.0%; Copper 2.0%

		Bondable		
Agility Convertible HG Gingival Buccal Tube	AC* AC1* AL1*	Prescription; Right, Left; Slot height; Weldable, Bondable	316 Stainless Steel, 302 Stainless Steel	Iron 60%; Chromium 18%; Nickel 14%; Molybdenum 2.25%; Manganese 2%

Warnings and Precautions

General		Devices are designed and manufacturer for single patient use only. The reuse or reprocessing may cause cross infection, loss of material or mechanical integrity and bond strength.
General		Avoid bonding tubes in occlusion to prevent wearing of enamel surfaces during all stages of treatment due to the hardness of metal tubes.
Bondable only		Bonding to compromised teeth can increase the risk of tooth damage.
Bondable only		Damage to the enamel of a tooth or to a restoration (crown, bonding, veneer, etc.) is possible when orthodontic appliances are removed.
General		Appliance breakage and or distortion require immediate attention by the orthodontist.
General		Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care.
General		Care should be taken to avoid habits or foods that could break the appliance. In the event of injury or ingestion, instruct the patient to seek medical care.
General		This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care.
General		This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required.

Clinical Benefits

Tubes, as a part of the orthodontic system, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

Orthodontic treatments are subject to relapse. Tubes, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications

Potential Side Effects or Adverse Events

- Allergic reaction, particularly if tube contains nickel.
- Discomfort and pain
- Oral and mucosal damage
- Ingestion
- Aspiration
- Adverse tooth movement resulting from wire distortion
- Tubes, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis

Instructions for use

The use of Orthodontic tubes is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient. The choice of prescription, slot size, convertability, weldability or bondability is made by the qualified professional, based on his knowledge acquired during his training in conjunction with the diagnosis and treatment plan for each patient.

1. Prepare the tooth surface according to the instructions of the manufacturer of the orthodontic adhesive.
2. Select the appropriate tube for the particular molar.
3. Position the tube on the tooth (or band) according to knowledge acquired during training in consideration of the diagnosis and treatment plan of the patient, coordinating with bracket placement.
4. **For bondable tubes:** Bond the tubes onto the molars according to the instructions of the manufacturer of the orthodontic adhesive.
5. **For weldable tubes:** Weld the tubes onto the band according to knowledge acquired during training. Place the band on the teeth using specific instruments, according to the manufacturer's instructions for use of the adhesive or cement purchased.
6. Engage the selected archwire.
7. Cut off the excess wire at the distal of the tube, or turn the end of the wire 90 degrees to the lingual (heat treatment optional according to orthodontist training), to prevent injuries in soft tissue of the patient.
8. After treatment, remove the bondable tubes or bands and orthodontic adhesive from the teeth using an appropriate instrument as per the orthodontic adhesive manufacturer's instructions for use.

To avoid ingestion or aspiration debond the tubes with the archwire in place and ligated.

9. Patient should be monitored regularly when undergoing orthodontic treatment. Particular care should be taken to monitor for wire distortion.

Maintenance and Storage instructions

Store tubes in product packaging until use.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions

	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care.
	Follow orthodontist recommendations for proper at home care.
	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Appliance breakage and or distortion requires immediate attention by the orthodontist.
	Care should be taken to avoid habits or foods that could break the appliance. In the event of injury or ingestion, instruct the patient to seek medical care.
	This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately

	remove orthodontic appliance and instruct the patient to seek medical care.
	This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required.

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number.

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