Orthodontic Headgear

Intended Use

Extraoral orthodontic headgear is a device intended to exert pressure on the teeth from outside the mouth. It is designed for intrusion and extrusion of molars, expansion and distalization.

Description of the device

The headgear has an outer bow that attaches to straps which are wrapped around the patient's neck or head and an inner bow portion intended to be fastened to the orthodontic appliance in the patient's mouth. The inner bow may include a bend in the leg called an Omega loop for adjustment purposes.

1. Part Numbers

Product	Part Number	Variants	Material
Cervical Neck Pads	CP****	Color	Fabric 90% polyester, 10% cotton
High Pull Headcaps	HP****	Color and Size	Nylon
Chin Cup Headcaps	SOF_***	Size	Polyurethane foam, nylon fabric, velcro tab, rubber based polyester filament elastic, and poly cotton blend thread.
Safety Release Modules	SR*****	Color and force range	Clasp: Chrome plated metal Housing: Polystyrene and Polycarbonate Strap: PVC / urethane Springs: Stainless Steel
Color Coated Facebows	CCFB***	Color Size of inner bow Universal – no preformed omega loop on inner bow	Stainless Steel AISI 302/AISI 304 and epoxy polyester
Stainless Steel Facebows	STFB*** M500***	Size of outer bow Size of inner bow Universal – no preformed omega loop on inner bow	Stainless Steel Polyethylene

Warnings and Precautions

General		Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
		Avoid sports activities when wearing orthodontic headgear; injury to face or eyes can occur.
General		Appliance breakage and or distortion require immediate attention by the orthodontist.
Facebows Only		This product contains nickel, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care.

Clinical Benefits

Headgear, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

Orthodontic treatments are subject to relapse.

Potential Side Effects or Adverse Events

- Laceration within the oral cavity
- Broken teeth
- Facial injury
- Eye injury

Instructions for use

The use of Orthodontic headgear is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

G&H recommends using facebows only with properly designed safety release systems to prevent potential harm to patients.

1. Select the suitable size.
2. Ensure that the inner bow has sufficient distance from the anterior teeth.
3. Guide the ends of the inner bow into the buccal tubes and attach safety straps onto the ends of the outer bow.
4. Patient should be monitored regularly when undergoing orthodontic treatment.

Maintenance and Storage instructions

Clean with mild soap and water, as needed. Store in Headgear storage case or pouch.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions


General	Clean with mild soap and water, as needed. Store in Headgear storage case or pouch.
General	Appliance breakage and or distortion require immediate attention by the orthodontist.
Facebows Only	Clean with toothpaste and a toothbrush.
General	Always insert the distal ends into the tubes first and then hook up the pulling mechanism.
General	When removing, always release the pulling mechanism first, then pull the face bow out of the tubes.
General	Do not allow other persons to touch or wear the equipment supplied.
General	Never pull in front of the eyes.
General	Do not use during sports.
General	Care should be taken not to bend outer or inner bow.

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the

inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

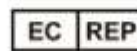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