Orthodontic Reverse Pull Face Masks

Intended Use

An extraoral orthodontic headgear is a device intended to exert pressure on the teeth from outside the mouth. It is designed for intrusion and extrusion of molars, expansion and distalization.

Description of the device

The headgear has an outer bow that attaches to straps which are wrapped around the patient's neck or head and an inner bow portion intended to be fastened to the orthodontic appliance in the patient's mouth.

1. Part Numbers

Product	Part	Variants	Material
Reverse Pull Face Mask	M540* RPFM*	Size	Polyethylene Stainless Steel 303 Aluminum 5052

Warnings

	Devices are designed and manufacturer for single orthodontics use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	NICKEL WARNING: This product contains nickel, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care.
	Avoid sports activities when wearing orthodontic headgear. Injury can occur.

Clinical Benefits

The main benefits that these systems bring to the life of the patient is the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

All finished orthodontic treatments are subject to relapse. After treatment, the control must be determined by the professional to avoid or minimize relapse.

Potential Side Effects or Adverse Events


The following have been reported as being associated with poor oral hygiene

- Development of plaque
- Halitosis
- Overall periodontal status

Instructions for use

The use of Orthodontic headgear is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

1. Select the suitable size for the patient.
2. Use the screwdriver include in the package to unscrew the fixation screws.
3. Place the parts of the structure in the desired position and fasten the screws.
4. For better adjustment of frontal support on the patient, it is possible to change its shape by: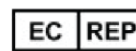

- a. Unfasten the screws and remove the upper part of the structure of the facemask.
 - b. Put it in warm water (50°-60°C/122°-140°F) for 3 minutes max.
 - c. Push the extremities of the frontal support, making the desired curved shape.
 - d. Keep this curved shape and put in cold water (room temperature).
 - e. Dry the support and assemble the rest of the facemask structure.
5. Apply the adhesive pads on frontal and chin support.
 6. Place the facemask on the patient.

Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacture and the competent authority.

Maintenance and Storage instructions

Must be stored in dry place at room temperature (15°-30°C max / 59°-86°F max) and protected from sunlight. May be stored in its own package.

Clean with mild soap and water, as needed.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions


Do not use during sports.

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please

contact HMI™ Customer Service for a return authorization number.

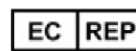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