Orthodontic Hooks (Weldable or Bondable)

Intended Use

To provide a method to transfer force from a wire, spring, elastic, or other force containing device to a tooth in order to facilitate orthodontic treatment.

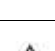
Description of the device

Items within the orthodontic hooks product family include small, inactive devices used to connect to other orthodontic appliances, or fastened to teeth to be connected to by other orthodontic appliances, to facilitate orthodontic treatment.

1. Part Numbers

| Product | Part | Variants | Material | Composition |
|---|-----------------------------------|----------|-----------------|---|
| Hooks Stainless Steel | M005* M006* M024* SURHK* | | Stainless Steel | |
| Ball Clasps Stainless Steel | SSBC* | | Stainless Steel | |
| Bondable Lingual Cleats/Sheaths Stainless Steel | M015B* | | Stainless Steel | |
| Bondable Lingual Buttons | M021B* | | Stainless Steel | Iron 65-75 Chromium 16-20 Nickel 8-13 Molybdenum 2.0-2.5 |

Warnings and Precautions

| | | |
|---------------|---|--|
| General |  | Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product. |
| |  | Avoid bonding tubes in occlusion to prevent wearing of enamel surfaces during all stages of treatment due to the hardness of metal tubes. |
| Bondable only |  | Bonding to compromised teeth can increase the risk of tooth damage. |
| Bondable only |  | Damage to the enamel of a tooth or to a restoration (crown, bonding, veneer, etc.) is possible when orthodontic appliances are removed. |
| |  | Appliance breakage and or distortion require immediate attention by the orthodontist. |
| |  | Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care. |
| |  | Care should be taken to avoid habits or foods that could break or distort the hook. In the event of injury or ingestion, instruct the patient to seek medical care. |
| |  | This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care. |

This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required.

Clinical Benefits

Hooks, as a part of the orthodontic system, bring an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

Orthodontic treatments are subject to relapse. Hooks, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications

Potential Side Effects or Adverse Events

- Allergic reaction, particularly if band contains nickel.
- Discomfort and pain
- Oral and mucosal damage
- Gingival impingement
- Ingestion
- Aspiration
- Hooks, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis.

Instructions for use

The use of Orthodontic hooks is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient. The type of hook to be used is made by the qualified professional, based on their knowledge acquired during their training in conjunction with the diagnosis and treatment plan for each patient.

1. Select the hook to be used.
2. **If welding a hook:** weld onto the band according to knowledge acquired during training. Cement band according to instructions in Band IFU.
3. **If bonding a hook:**
4. Prepare the tooth surface according to the instructions of the manufacturer of the orthodontic adhesive.
5. Apply adhesive to the hook according to the instructions of the manufacturer of the orthodontic adhesive.
6. Position and bond the hook on the tooth according to knowledge acquired during training in consideration of the diagnosis and treatment plan of the patient.
7. The hook is now accessible as a local anchor point for wires, thread, springs, elastics or string.
8. After treatment, remove the hooks and orthodontic adhesive from the teeth using an appropriate instrument as per the orthodontic adhesive manufacturer's instructions for use.
9. Patient should be monitored regularly when undergoing orthodontic treatment. Particular care should be taken to monitor for band loosening or damage.

Any serious incident experienced with the use of the product should be reported to the manufacture and the competent authority.

Maintenance and Storage instructions

Store hooks in product packaging until use.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions

| | |
|---|--|
|  | Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care. |
|  | Follow orthodontist recommendations for proper at home care. |
|  | Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease. |
|  | Appliance breakage and or distortion requires immediate attention by the orthodontist. |
|  | Care should be taken to avoid habits or foods that could break the appliance. In the event of injury or ingestion, instruct the patient to seek medical care. |
|  | This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care. |
|  | This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required. |

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

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