Orthodontic Elastics

Intended Use

Orthodontic elastics are used in orthodontic treatment to exert a force.

Description of the device

Elastics are elastic bands cut from tubing. Elastics are connected to attachments as part of an orthodontic appliance or aligner trays. Elastics are also used to connect extra-oral orthopedic appliances. Elastics are available in both latex and non-latex.

1. Part Numbers

Product	Part	Variants	Material	Composition
Intraoral Latex Elastics	AM* OC* 14*AM 18*AM 36*AM 38*AM 56*AM M931*	Band Size: 1/4", 1/8", 3/16", 3/8", 5/16"; Band Force: Light, Medium, Heavy, Extra Heavy	K-100 Natural Rubber Latex	Natural Rubber 95-97%, Sulfur <1%, Zinc Oxide <1%, Polymetric Hindered Phenol <2%, Carbamate Derivative <1%
	Extraoral Latex Elastics			
Intraoral Colored Latex Elastics	AC* OC*C 14*AC 18*AC 36*AC 56*AC	Band Size: 1/4", 1/8", 3/16", 5/16"; Band Force: Medium, Heavy	K-100 Latex Colored	Natural Rubber Latex 90-95%, Colorant Encapsulated in Rubber <7%, Vulcanizing Agents 3-5%

Warnings and Precautions

	Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	Avoid sports activities when wearing orthodontic headgear; injury to face or eyes can occur. In the event of injury, however minor, instruct the patient to seek medical care.
	Care should be taken to avoid habits or foods that could break or dislodge the elastics. Follow orthodontist recommendations for proper at home care. In the event of injury or ingestion, instruct the patient to seek medical care.
	This product contains natural rubber latex which may cause allergic reactions.

Clinical Benefits

Elastics, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

Residual Risks

Orthodontic treatments are subject to relapse. Elastics, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications.

Potential Side Effects or Adverse Events

- Allergic reaction
- Irritation to the skin
- Facial injury, when used with facemasks
- Eye injury, when used with facemasks
- Discomfort and pain
- Oral and mucosal damage
- Gingival impingement
- Ingestion
- Aspiration
- Elastics, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis.

Instructions for use

The use of Orthodontic elastics is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient. The type of elastic to be used is made by the qualified professional, based on their knowledge acquired during their training in conjunction with the diagnosis and treatment plan for each patient.

1. Select the elastic band to be used based on desired force. Consider latex allergy.

	Consider latex allergy when selecting the appropriate elastic band.
--	---

2. Connect the elastic to the appropriate points of the orthodontic appliance to create a force vector as required.
3. Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacture and the competent authority.

Maintenance and Storage instructions


Store elastic bands in product packaging until use.

Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

Patient Instructions

	Elastic rubber bands should be replaced according to the instructions provided by the dental professional.
	Patient compliance and full understanding of proper band placement is required for successful outcome.
	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Follow orthodontist recommendations for proper at home care. Care should be taken to avoid habits or foods that could break or dislodge the elastic band. In the event of breakage, reapply a new elastic band according to the orthodontist instructions.

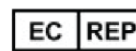
When used with extraoral headgear, follow the instructions provided by the dental professional.

Warranty

Limited Warranty. Product(s) manufactured by G&H are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. G&H's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in G&H's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of G&H product to which this warranty applies, G&H determines that a claimed defect was not due to its manufacture or materials, G&H will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to G&H products, or distributed by G&H to customer, are not warranted by G&H but bear only the express warranty, if any, of the manufacturers thereof. No representative of G&H has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of G&H. G&H's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in G&H's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. G&H will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, G&H products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

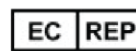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