**Orthodontic Elastomeric Thread**

**Intended Use**

Elastomeric Threads, Hollow, Solid and Slip-Not are used to tie appliances and auxiliaries into place.

**Description of the device**

Elastomeric Threads, Hollow, Solid and Slip-Not are continuous lengths of elastomeric “thread” from extruded products. The Hollow is hollow; the Slip-Not thread is solid and has a textured surface to facilitate tying.

**1. Part Numbers**

Product	Part Number	Variants (size)	Material
Elastomeric Thread	TB* ET* SNT*	Color, size, solid or hollow, surface finish	Texin 285

**Warnings and Precautions**

	Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	Thread breakage and or dislodgement requires immediate attention by the orthodontist.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur.
	Care should be taken to avoid habits or foods that could break or dislodge the thread. Follow orthodontist recommendations for proper at home care.

**Clinical Benefits**

Elastomeric thread, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

**Residual Risks**

Orthodontic treatments are subject to relapse. Elastomeric thread, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications.

**Potential Side Effects or Adverse Events**

- Discomfort and pain
- Gingival impingement
- Ingestion
- Aspiration
- Elastomeric thread, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis.

**Instructions for use**

The use of elastomeric thread is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

1. Select the elastomeric thread type.

2. Cut appropriate length for tying.
3. Tie auxiliaries to each other, brackets or wire. When securing elastomeric thread to a bracket, it is recommended to secure the thread under the bracket.
4. Shorten or cut the excess end and manipulate it away from the mucosa, lip or cheek.
5. Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacturer and the competent authority.

**Maintenance and Storage instructions**

Store elastomeric thread in product packaging until use.

**Disposal**

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

**Patient Instructions**

	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur.
	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Elastomeric thread breakage and or dislodgement requires immediate attention by the orthodontist.
	Care should be taken to avoid habits or foods that could break or dislodge the elastomeric thread. Follow orthodontist recommendations for proper at home care.

**Warranty**

Limited Warranty. Product(s) manufactured by Flexmedics are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. Flexmedic’s obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in Flexmedic’s sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of Flexmedics product to which this warranty applies, Flexmedics determines that a claimed defect was not due to its manufacture or materials, Flexmedics will invoice customer for the value of product and services at customer’s expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to Flexmedics products, or distributed by Flexmedics to customer, are not warranted by Flexmedics but bear only the express warranty, if any, of the manufacturers thereof. No representative of Flexmedics has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of Flexmedics. Flexmedic’s liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in Flexmedic’s sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. Flexmedics will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, Flexmedic products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original

packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

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