**Orthodontic Elastomeric Chain**

**Intended Use**

Orthodontic elastics are used in orthodontic treatment to exert a force. Elastomeric chain is placed around the tie wings on the bracket to retain the archwire in the bracket slot and extended to other teeth to provide force.

**Description of the Device**

Elastomeric Chain is continuous length of toric-shaped elastomers.

**1. Part Numbers**

Product	Part Number	Variants (size)	Material
Elastomeric Chain	CDL* LDL* SDL*	Spacing, thickness and color	Texin 285
Powerlinx Chain	LPL* SPL*	Spacing and color	

**Warnings and Precautions**

	Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	Chain breakage and or dislodgement requires immediate attention by the orthodontist.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur.
	Care should be taken to avoid habits or foods that could break or dislodge the chain. Follow orthodontist recommendations for proper at home care.

**Clinical Benefits**

Elastomeric chain, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

**Residual Risks**

Orthodontic treatments are subject to relapse. Elastomeric chain, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications.

**Potential Side Effects or Adverse Events**

- Discomfort and pain associated with wire engagement
- Gingival impingement
- Ingestion
- Aspiration
- Elastomeric chain, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis.

## Instructions for Use

The use of elastomeric chain is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient.

1. Select the elastomeric chain considering desired spacing between the links and thickness. Force generation is influenced by the number of links between adjacent teeth. Consider patient preferences when choosing color.
2. Place the elastomeric chain over the tie wings of the brackets until the desired securement has been achieved.
3. Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacturer and the competent authority.

## Maintenance and Storage Instructions

Store elastomeric chain in product packaging until use.

## Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

## Patient Instructions

	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur.
	Elastomeric chain breakage and or dislodgement requires immediate attention by the orthodontist.
	Care should be taken to avoid habits or foods that could break or dislodge the elastomeric chain. Follow orthodontist recommendations for proper at home care.

## Warranty

Limited Warranty. Product(s) manufactured by Flexmedics are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. Flexmedic's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in Flexmedic's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of Flexmedics product to which this warranty applies, Flexmedics determines that a claimed defect was not due to its manufacture or materials, Flexmedics will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to Flexmedics products, or distributed by Flexmedics to customer, are not warranted by Flexmedics but bear only the express warranty, if any, of the manufacturers thereof. No representative of Flexmedics has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of Flexmedics. Flexmedic's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in Flexmedic's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. Flexmedics will not be liable for any other injury, loss, damage or expense, whether

direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, Flexmedics products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

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