**Orthodontic Hooks and Stops (Wire or Bracket Attachments)**

**Intended Use**

Hooks and stops provide a method to transfer force from a wire, spring, elastic, or other force containing device to a tooth in order to facilitate orthodontic treatment.

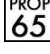
**Description of the Device**

Items within the hooks and stops product family include small, inactive devices used to connect to other orthodontic appliances by sliding or crimping onto the archwire to facilitate orthodontic treatment.

**1. Part Numbers**

Product	Part	Variants	Material
Slide On Vertical Tubes	V-T*	Slot size	Stainless Steel
Crimpable Stops	CSTOP* M022*	Wire size compatibility and shape	Stainless Steel

**Warnings and Precautions**

	Devices are designed and manufactured for single patient use only. The reuse or reprocessing may cause cross infection or loss of mechanical properties due to natural wear of the product.
	Appliance breakage and or distortion require immediate attention by the orthodontist.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or distortion of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care.
	Care should be taken to avoid habits or foods that could break or dislodge the hook or stop. Follow orthodontist recommendations for proper at home care. In the event of injury or ingestion, instruct the patient to seek medical care.
	This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately remove orthodontic appliance and instruct the patient to seek medical care.
	This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required.

**Clinical Benefits**

Hooks and stops, in conjunction with orthodontic appliances, brings an aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

**Residual Risks**

Orthodontic treatments are subject to relapse. Hooks and stops, as part of the fixed orthodontic system, can contribute to discoloration or decalcification of teeth, bone and root resorption, and periodontal complications.

**Potential Side Effects or Adverse Events**

- Allergic reaction, particularly if hook contains nickel.
- Discomfort and pain
- Oral and mucosal damage
- Gingival impingement
- Ingestion
- Aspiration
- Hooks and stops, as part of the fixed orthodontic system, can contribute to difficulties in maintaining oral hygiene, difficulty speaking or chewing and halitosis.

### Instructions for use

The use of Orthodontic hooks and stops is restricted to dental professionals with qualification in orthodontics. Use by persons outside this context may result in irreversible damage to the patient. The type of hook or stop to be used is made by the qualified professional, based on their knowledge acquired during their training in conjunction with the diagnosis and treatment plan for each patient.

1. Select the hook or stop to be used.

Note:	Best results are achieved when a crimping plier is used, especially for hooks.
-------	--

2. For closed-walled hooks or stops, slide the hook or stop onto the archwire and crimp to secure.
3. For open-walled hooks or stops, archwire does not have to be removed. Place the hook or stop directly onto the archwire at desired location and crimp to secure.
4. For slide on vertical tube, slide the tube on the archwire to desired location. Place shepherd hook or "T" Hook into the v-slot tube, as applicable.
5. For brackets with integrated vertical slots place shepherd hook or "T" Hook into the v-slot of the bracket and secure.
6. Patient should be monitored regularly when undergoing orthodontic treatment.

Any serious incident experienced with the use of the product should be reported to the manufacturer and the competent authority.

### Maintenance and Storage instructions

Store hooks and stops in product packaging until use

### Disposal

The professional should use good clinical practices for the disposal of medical products in accordance with current sanitary regulations.

### Patient Instructions

	Practice good oral hygiene to prevent plaque installation and subsequent periodontal disease.
	Patients should wear a mouthguard when engaging in sports activities; injury to the patient and /or breakage or dislodgement of the appliance can occur. In the event of injury or ingestion, instruct the patient to seek medical care.
	Appliance breakage and or dislodgement requires immediate attention by the orthodontist.
	Care should be taken to avoid habits or foods that could break or distort the hook. Follow orthodontist recommendations for proper at home care. In the event of injury or ingestion, instruct the patient to seek medical care.
	This product contains nickel or chromium, a chemical known to the State of California to cause cancer. In the event of an allergic reaction, immediately

	remove orthodontic appliance and instruct the patient to seek medical care.
	This medical device is MR Conditional. Scanning of the head and neck may cause image artifacts. If an MRI is needed, contact the radiologist prior to the procedure for instructions regarding appliance removal, if required.

## Warranty

Limited Warranty. Product(s) manufactured by FlexMedics are warranted to be free of defects in materials and manufacture for ninety (90) days from date of delivery. Any material or manufacture defect covered by this limited warranty which occurs during normal use and is reported to Seller in writing during the period of ninety (90) days from the date such products are shipped to customer. FlexMedic's obligation hereunder, upon verification of the defect or error, shall be to provide one of the following, in FlexMedic's sole discretion: (i) replacement at no charge to customer; or (ii) credit the purchase price to customer. If, upon the inspection of FlexMedics product to which this warranty applies, FlexMedics determines that a claimed defect was not due to its manufacture or materials, FlexMedics will invoice customer for the value of product and services at customer's expense. This warranty shall not apply to any product that has been subject to misuse, misapplication, neglect (including but not limited to use of unauthorized parts or attachments), adjustment or repair and/or improper handling and storage. Identifiable items manufactured by others but installed in or affixed to FlexMedics products, or distributed by FlexMedics to customer, are not warranted by FlexMedics but bear only the express warranty, if any, of the manufacturers thereof. No representative of FlexMedics has any authority to waive, alter, vary or add to the terms hereof without prior approval in writing, to customer, signed by an officer of FlexMedics. FlexMedic's liability for its products, whether for breach of contract, negligence, strict liability in tort, or otherwise, shall be limited, in FlexMedic's sole discretion, to the replacement of the products or parts thereof or a credit to the purchase price to customer. FlexMedics will not be liable for any other injury, loss, damage or expense, whether direct or consequential, including but not limited to loss of use, income, profit or production, or increased cost of operation, or spoilage of or damage to material, arising in connection with the sale, installation, use of, inability to use, or the repair or replacement of, or late delivery of, FlexMedics products.

Limited Warranty. Product(s) manufactured by Highland Metals are warranted to be free from defects. Products must be returned within 90 days from the date of purchase and must be returned in original packaging and in resalable condition. HMI™ reserves the right to final determination as to acceptance of product exchange or credit. Restock fee may apply. Prior to the return of any merchandise, please contact HMI™ Customer Service for a return authorization number

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